

Technical Information

Accuracy: +0.1 -0.15°C

Resolution: ±0.1°C



Brannan Thermometers
Cleator Moor
Cumbria CA25 5QE
England

Instructions for use

- Use light disinfectant to clean the thermometer before use.
- Ensure the thermometer is reset: hold the thermometer at the top and shake vigorously at arms length.
- Place the thermometer under the tongue.
- Leave the thermometer under the tongue for **at least four minutes**. [too short a time may result in an incorrect reading]
- After four minutes, remove and read the thermometer.
- The generally accepted normal body temperature for a healthy person is 37°C. However there are measurable differences between individuals.

Warning *As the device is placed in the mouth, there is a risk of breakage of the product by the user in certain circumstances; this could lead to lacerations caused by broken glass and possible ingestion of parts of the device: glass and mercury. The device should not be used with young children and patients suffering from epileptic fits or otherwise distressed.*