

Technical Information

Accuracy: $+0.1 -0.15^{\circ}\text{C}$
Resolution: $\pm 0.1^{\circ}\text{C}$



Brannan Thermometers
Cleator Moor
Cumbria CA25 5QE
England

exp:

Rectal Thermometer Instructions for use

- This rectal thermometer is marked with a blue dot to differentiate it from thermometers for oral use.
- Use light disinfectant to clean the thermometer before use.
- Ensure the thermometer is reset: hold the thermometer at the top and shake vigorously at arms length.
- Place the thermometer in the rectum.
- Leave the thermometer in the rectum for at least four minutes. [too short a time may result in an incorrect reading] (as recommended by BS 691)
- After four minutes, remove the thermometer and read the temperature.
- The generally accepted normal body temperature for a healthy person is 37°C . However there are measurable differences between individuals.

Warning

As the device is placed in the rectum, there is a risk of breakage of the product by the user in certain circumstances; this could lead to lacerations caused by broken glass and possible retention of parts of the device: glass and mercury.

This device should only be used by trained medical operatives.