

Product data sheet

✉ sales@brannan.co.uk

☎ +44 (0)1946 816600

🌐 brannan.co.uk

ADVANCED DIGITAL VETERINARY THERMOMETER



Brannan Veterinary Digital Thermometer								
Hobday's Specification	Man	Cat	Horse	Dog	Cow	Pig	Sheep	Powl
	37	38	38	38.8	38.9	39.5	40	41.7 °C

Instructions for Use

1. Press the on/off button. The display will read 188.8°C.
2. Release the power switch and the display will show L°C with °C flashing.
3. Position the probe.
4. Once the degree sign on the display has stopped flashing (usually within 30 to 60 seconds), the correct temperature is indicated. The temperature reading will not change after the °C stops flashing. An alarm signal will sound for approximately 10 seconds when °C stops flashing.
5. Remove the thermometer, being careful to avoid pressing the on-off button. The temperature may be read before removal because the display will retain the stabilized temperature reading.
6. The unit will automatically turn off in 8-10 minutes. However, to prolong battery life, press the power switch once the temperature has been noted to turn off thermometer.

Battery Replacement

When the battery is almost exhausted, the following symbol "🔋" will appear in the display window.

1. Remove battery cover as shown in illustration.
2. Use toothpick or insulated probe to remove battery. Avoid using any sharp metal object in this operation.
3. Insert new battery with positive side facing upwards.
4. Replace and fasten the cover securely.

WARNING: Do not attempt to recharge battery. Do not dispose of battery in or near fire as it may explode. Swallowing a button battery may be fatal.

Brannan Thermometers & Gauges • Cleator Moor • Cumbria • CA25 5QE • UK
tel: +44 (0)1946 816600 • fax: +44 (0)1946 816625 • web: www.brannan.co.uk • e-mail: sales@brannan.co.uk

Easy to use waterproof, electronic thermometer with a flexible probe and high speed response for quick and accurate reading. Supplied with chart giving Hobday's specification (the normal body temperature for a variety of animals).

Product features:

- Compact, accurate and durable
- Fast response, waterproof thermometer
- Non hazardous
- Complete with battery, instructions and Hobday chart

Product specifications

Temperature range:	32°C to 42.9°C
Ingress protection:	IP22
Accuracy:	+/-0.1°C between 35.5°C and 42.9°C, otherwise +/-0.2°C
Resolution:	0.1°C
Approx response time:	60 seconds
Expected lifespan:	Approx 5 years
Battery:	1 x LR41 or equivalent (included)
Approx battery life:	100 hours continuous use (2+ years when idle)
Storage temperature:	-25°C to +55°C / <95% RH
Operating temperature:	+5°C to +35°C / <80% RH
CE & RoHS/WEEE:	Compliant
Hazard information (SDS):	See brannan.co.uk for information
LCD dimensions:	7mm x 20mm
Product dimensions:	Net: 133mm x 22mm x 12mm Gross: 167mm x 56mm x 18mm
Product weights:	Net: 19g (including batteries and probe case) Gross: 25g (including chart)

View me on the website



Description	Barcode	Product No
Advanced digital veterinary thermometer	5 011405 110350	11/035/2

Measurement for life.

